



- APPROPRIATE SETTINGS FOR THE INSERTS
- SETTAGGI APPROPRIATI PER GLI INSERTI
- KORREKTE EINSTELLUNGEN FÜR DIE INSTRUMENTE
- RÉGLAGES APPROPRIÉS POUR LES INSERTS
- AJUSTES APROPIADOS PARA LOS INSERTOS
- KORREKTA INSTÄLLNINGAR FÖR SPETSARNA

→ MULTIPIEZO PRO → MULTIPIEZO touch → MULTIPIEZO white



function	endo	perio/scaler	restorative/prosthetics
----------	------	--------------	-------------------------

SCALING

S1-S		1-2 **	
S1, S2, S3, S4, S5, S6, S7, S8		1-3 **	

PERIO

P1, P2, P3, P4, P10, P11, P12, P13, P14		1-3 **	
P15, P16R, P16L		1-2 *	

IMPLANT CLEANING

ICS		1-2 *	
-----	--	-------	--

ENDO

E1, E2	1-6		
--------	-----	--	--

ENDO REVISION

ER1		2-4	
ER2, ER3, ER4, ER5		1-3	

ENDO RETRO

R1, R2, R3, R4, R5, RD3, RD4, RD5		1-3	
-----------------------------------	--	-----	--

CROWN PREP

DB1			4-6
-----	--	--	-----

CAVITY MARGIN

CM1, CM2, CM3, CM4			2-5
--------------------	--	--	-----

RESTORATIVE

D1, D2, D4			1-6
D3			4
ME1, ME2, ME3 ***			6

* it is recommended the use of „soft mode“ function

** can be used with or without „soft mode“ function

*** not available in the United States and in Italy



- APPROPRIATE SETTINGS FOR THE INSERTS
- SETTAGGI APPROPRIATI PER GLI INSERTI
- KORREKTE EINSTELLUNGEN FÜR DIE INSTRUMENTE
- RÉGLAGES APPROPRIÉS POUR LES INSERTS
- AJUSTES APROPIADOS PARA LOS INSERTOS
- KORREKTA INSTÄLLNINGAR FÖR SPETSARNA



function	endo	perio/scaler	restorative	power	power
----------	------	--------------	-------------	-------	-------

SCALING

S1-S		1-2 **		1-2	1-2
S1, S2, S3, S4, S5, S6, S7, S8		1-3 **		1-3	1-3

PERIO

P1, P2, P3, P4, P10, P11, P12, P13, P14		1-3 **		1-3	1-2
P15, P16R, P16L		1-2 *		2	2

IMPLANT CLEANING

ICS		1-2 *		1-2	1-2
-----	--	-------	--	-----	-----

ENDO

E1, E2	1-5			1-2	1
--------	-----	--	--	-----	---

ENDO REVISION

ER1		1-5		3-6	2-4
ER2, ER3, ER4, ER5		1-3		1-3	1-2

ENDO RETRO

R1, R2, R3, R4, R5, RD3, RD4, RD5		1-3		1-4	1-3
-----------------------------------	--	-----	--	-----	-----

CROWN PREP

DB1			4-5	4-6	3-4
-----	--	--	-----	-----	-----

CAVITY MARGIN

CM1, CM2, CM3, CM4			1-4	1-5	1-3
--------------------	--	--	-----	-----	-----

RESTORATIVE

D1, D2, D4			1-4	4-6	3-4
D3			1-4	3-5	2-3
ME1, ME2, ME3 ***			5		

* it is recommended the use of „soft mode“ function

** can be used with or without „soft mode“ function

*** not available in the United States and in Italy

